

1. Volume control for handset or speaker phone.
2. Speaker Phone button and the Cancel button to end that function. **Note: You must hit the Cancel button to end conversation with the patient if you use the speaker phone.**
3. Scroll button for the soft keys. Note: You have two rows.
4. Soft Keys. **Need Nurse Or Need Tech.** (Push while in communication to room to set a **red** reminder light or **orange** for tech). **Night Tones** (That will reduce call volume my 30%). **Calls to console will allow calls to annunciate at desk.**
5. Incoming patient call touch points. Note: If you pick up the handset, it will pick up the top line. You must select line then pick-up the handset if you don't want the top call.
6. Incoming patient call scroll button. This will get you to addition patient calls that are not showing.



Look at Page 2 to see the remaining Soft keys

1. **Room Volume (+)** While in communication to the room, you can increase the volume to the room regarding what the patient can hear from you.
2. **Room Volume (-)** While in communication to the room, you can decrease the volume to the room regarding what the patient can hear from you.
3. **All Page** – Annunciates a voice message to all patient rooms on unit
4. **Mute Tones** (This will silence the incoming call for 2 mins.

Note: The Volume is defaulted to 5 and the range is from 1 through 10 ++ which is like 12. 5 is usually loud enough.




Enhanced Pillow Speaker

1. Water – Call sent to Nurse wireless phones and the Nurse Console in 60 sec. (Solid White & Green Light outside room) Cancel upon answer or station cancel.
2. Pain – Call sent to Nurse wireless phones and the Nurse Console in 60 sec. (Solid White & Red Light outside room). Cancel upon answer or station cancel.
3. Toilet - Call sent to Nurse wireless phones and the Nurse Call in 60 sec. (Solid White & Green Light outside room) Cancel upon answer or station cancel.
4. Nurse Button – Call sent to Nurse wireless phones and the Nurse Console in 60 sec. (Solid White Light outside room) Cancel upon answer or station cancel.
5. General Buttons to change channel, Close Caption, Mute the TV, and Turn on/off lights behind or above patient.




Enhanced Single Patient Station NICU

1. Staff Assist – Annunciates to all wireless phones, Clinical Advisor Team, & Nurse Call Console . 4 Flashing Red Lights & higher ring tone. Station Cancel only.
2. Code Blue or Neo Code– Annunciate to all on-duty wireless devices on unit, Code Team, and staff/duty stations if applicable. 4 Strobing Blue Light & higher ring tone. Station cancel only.
3. Pillow Speaker plug-in - Pillow Speaker plugs in here.
4. Aux Jack – Defaulted Patient Call for another call cord or specialty call cord. Same workflow as patient call. (1 Solid White Light Outside Room)
5. Cancel Button – Cancels Bed Exit Alarm, Staff Assist Calls, Code Blue Calls, All calls from pillow speaker.

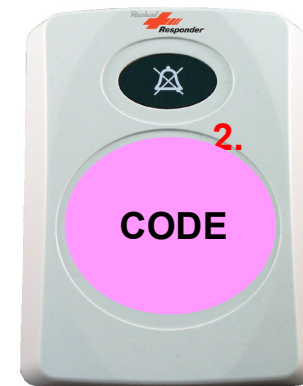


Enhanced Single Patient Station MB

1. Staff Assist – Annunciates to all wireless phones, Clinical Advisor Team, MB Code Blue & Staff Assist Teams, & Nurse Call Console . 4 Flashing Red Lights & higher ring tone. Station Cancel only.
2. Code Blue or Neo Code– Annunciate to all on-duty wireless devices on unit, Code Team, MB Code Blue & Staff Assist Teams, and staff/duty stations if applicable. 4 Strobing Blue Light & higher ring tone. Station cancel only.
3. Pillow Speaker plug-in - Pillow Speaker plugs in here.
4. Aux Jack – Defaulted Patient Call for another call cord or specialty call cord. Can be programmed for other device like a Vent or IV pump. (1 Solid White Light Outside Room)
5. Cancel Button – Cancels Bed Exit Alarm, Staff Assist Calls, Code Blue Calls, All calls from pillow speaker.

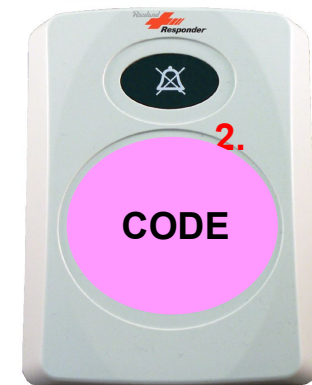

Enhanced Single Patient Station LD

1. Staff Assist – Annunciates to all wireless phones, Clinical Advisor Team, & Nurse Call Console . 4 Flashing Red Lights & higher ring tone. Station Cancel only.
2. Code Blue or Neo Code – Annunciate to all on-duty wireless devices on unit, Code Team, and staff/duty stations if applicable. 4 Strobing Blue Light & higher ring tone. Station cancel only.
3. Pillow Speaker plug-in - Pillow Speaker plugs in here.
4. Aux Jack – Defaulted Patient Call for another call cord or specialty call cord. Can be programmed for other device like a Vent or IV pump. (1 Solid White Light Outside Room)
5. Cancel Button – Cancels Bed Exit Alarm, Staff Assist Calls, Code Blue Calls, All calls from pillow speaker.

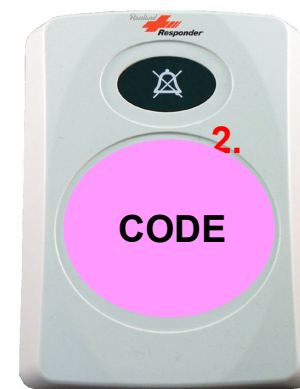

NICU 1st Set

1. Aux 1– Annunciates **Monitor** call to Nurse wireless device , then the Charge Wireless, and the Nurse Call Console in 30 secs. Cancel button to Cancel. 1 Flashing White light.
2. Aux 2– Annunciates **Vent** call to Nurse wireless device , then the Charge Wireless, and the Nurse Call Console in 30 secs. Cancel button to Cancel. 4 Red/White Flashing Lights.

NICU 2nd Set

1. Aux 1– Annunciates **Vent 2** call to Nurse wireless device , then the Charge Wireless, and the Nurse Call Console in 30 secs. Cancel button to Cancel. 4 Red/White Flashing Lights.
2. Aux 2– Annunciates **Equipment** call to Nurse wireless device , then the Charge Wireless, and the Nurse Call Console in 30 secs. Cancel button to Cancel. 1 Blue/1 yellow Flashing Lights.



MB & LD 1st Set

1. Aux 1– Annunciates **Chair Exit** call to all wireless device right away, and the Nurse Call Console. Cancel button to Cancel. 4 Flashing Yellow lights.
2. Aux 2– Annunciates **Monitor** call to Nurse wireless device , then the Charge Wireless, and the Nurse Call Console in 30 secs. Cancel button to Cancel. 1 Flashing White light.

MB & LD 2nd Set

1. Aux 1– Annunciates **Vent 2** call to Nurse wireless device , then the Charge Wireless, and the Nurse Call Console in 30 secs. Cancel button to Cancel. 4 Red/White Flashing Lights.
2. Aux 2– Annunciates **Equipment** call to Nurse wireless device , then the Charge Wireless, and the Nurse Call Console in 30 secs. Cancel button to Cancel. 1 Blue/1 yellow Flashing Lights.



37 Pin Feature Bed Receptacle

1. 37 Bed Pin Feature Bed Receptacle–
Annunciates any Patient Nurse Call
Buttons that could be on side rail (Solid
White light outside room), when the bed
cord gets unplugged from this jack (2
Blinking White lights outside room), and will
annunciates any bed exit alarm (if bed exit
alarm is set on bed and will annunciate 4
Flashing yellow lights outside room).



Bath Audio Station

1. Bathroom Pull for Help Cord – Annunciates Bath Emergency to All On Duty Staff and the Console. 2 flashing red lights outside room.
2. Bathroom Cancel Button.
3. Bath Emergency – – Annunciates Bath Emergency to All On Duty Staff and the Console. 2 flashing red lights outside room.



Pull for Help Station (Shower)

1. Bathroom Pull for Help Cord – Annunciates Bath Emergency to All On Duty Staff and the Console. 2 flashing red lights outside room.
2. Bathroom Cancel Button.



Duty Station



353100

FEATURES

- Four (4) LED indicators mimic the corridor light behavior of a call in progress
- Call tone indication of calls are the same as at the nurse console(s)
- Pushbutton for two-way intercom in an audio-visual system with built-in Call-Assurance LED
- Speaker and microphone provide full-duplex audio to speak from station if desired
- Day/Night tone level control (from console) with tone mute for calls